# Puro™ vire

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## English

### What Puro™ Vire is and what it is intended for

This is a moisturizing eye drop product intended for relieving seasonal symptoms of irritation, dryness, and burning of the eyes. Evaporation and elevated osmolarity of the tear fluid damage the ocular surface, causing uncomfortable sensations in the eyes and blurring of vision. The active ingredient in Puro™ Vire is hyaluronan that binds water in the tear film, brings tear osmolarity back to a harmless level, and helps ocular surface to recover from damage and redness. The moisturizing and protective properties of hyaluronan relieve the symptoms of dry eye caused by environmental factors such as dry air, wind, smoke, air pollution, intensive light, as well as by ocular surgery, allergies, or contact lens wear. The composition of the moisturizing eye drops balances the tear fluid and promotes good vision, offering relief for strained eyes due to activities such as mobile device use, visual display work, reading, and driving. Puro™ Vire is a medical device product developed, manufactured, and clinically tested in Finland. Puro™ Vire comes without any preservatives and it is suitable for sensitive eyes and for improving the comfort of contact lens wear — the lenses need not be removed from the eyes. The bottle does not contain materials releasing silver ions into the eye.

### Instructions for use

Tear off and discard the seal of the cap to open the bottle for the first time. Open the cap. Practise handling of the bottle before the first application. Gently squeeze the bottle in tip down position only. Apply 1–2 drops in both eyes 2–3 times daily or as needed. Or, apply 1 drop on contact lens before inserting it on the eye and as needed during contact lens wear. Shake the remaining drop away from the tip and then close the cap. The contents should be used within six (6) months from opening and by the expiry date.

## Precautions and warnings

- Do not use, if you are sensitive to any of the ingredients.
- Do not use, if the bottle is damaged.
- The bottle should be used by one user only.
- Do not touch the eye or contact lens with the tip of the bottle. Do not touch the tip of the bottle with fingers.
- Do not squeeze the bottle with cap on.
- Do not use after expiry date printed on the bottle and carton box or after discard date (6 months after opening).
- If you experience blurring of vision just after application, please wait a moment before continuing activities that require clear vision. Blurring should be mild and temporary only.
- Wait at least 15 minutes between possible applications of other ophthalmic medication.
- Keep out of the reach and sight of children.
- Store at 5–30 °C. Protect from direct sunshine and freezing.
- Discard the empty bottle appropriately.

## Composition

0.2% hyaluronan in sterile isotonic, low-phosphate buffer solution. Contains no preservatives, alcohol, borates, mineral oil, or materials of animal origin.

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Recycle packing materials  
Sort box into carton waste container, leaflet into paper waste container and dispenser into plastic packaging waste container or energy waste.