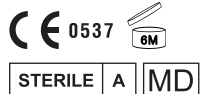
PuroTM teho

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English

What Puro™ Teho is and what it is intended for

This is an intensive moisturizing eye drop intended for relieving moderate and severe symptoms of dry eye. Evaporation and elevated osmolarity of the tear fluid in dry eyes damage the ocular surface, causing uncomfortable sensations in the eyes and blurring of vision. The active ingredients in Puro™ Teho are trehalose and hyaluronan that bind water in the tear film, bring tear osmolarity back to a harmless level, and help the ocular surface recover from damage and redness. The moisturizing, protective and lubricating properties of trehalose and hyaluronan relieve the symptoms of dry eye — irritation, itching, and foreign body sensation in the eyes — caused by environmental factors such as dry air, wind, smoke, air pollution, intensive light, as well as by ocular surgery, allergies, or contact lens wear. It is the high-molecular-weight hyaluronan that stabilises the tear film on the ocular surface and offers a long-lasting moisturizing effect. The composition of Puro™ Teho promotes good vision, relieving the strain of eyes due to activities such as mobile device use, visual display work, reading, and driving. Puro™ Teho is a medical device product developed and manufactured in Finland and with clinically tested active ingredients. It comes without any preservatives and it is particularly suitable for sensitive eyes and for improving the comfort of contact lens wear — the lenses need not be removed from the eyes. The bottle does not contain materials releasing silver ions into the eye.

Instructions for use

Tear off and discard the seal of the cap to open the bottle for the first time. Open the cap. Practise handling of the bottle before the first application. Gently squeeze the bottle in tip down position only. Apply 1–2 drops in both eyes 1–3 daily. Shake the remaining drop away from the tip and then close the cap. A small liner pad inside the cap ensures that the next drop you take will be fresh and clean. The contents should be used within six (6) months from opening and by the expiry date.

Precautions and warnings

- Do not use, if you are sensitive to any of the ingredients.
- Do not use, if the bottle is damaged.
- The bottle should be used by one user only.
- Do not touch the eye or contact lens with the tip of the bottle. Do not touch the tip of the bottle with fingers.
- Do not squeeze the bottle with cap on.
- Do not use after expiry date printed on the bottle and carton box or after discard date (6 months after opening).
- If you experience blurring of vision just after application, please wait a moment before continuing activities that require clear vision. Blurring should be mild and temporary only.
- Wait at least 15 minutes between possible applications of other ophthalmic medication.
- Keep out of the reach and sight of children.
- Store at 15–30 °C. Protect from direct sunshine and freezing.
- Discard the empty bottle appropriately.

Composition

2.0% trehalose, 0.25% high-molecular-weight hyaluronan in sterile isotonic, low-phosphate buffer solution. Contains no preservatives, alcohol, borates, mineral oil, or materials of animal origin.

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Recycle packing materials
Sort box into carton waste container, leaflet into paper waste container and dispenser into plastic packaging waste container or energy waste.